

High Plains Radiological Assn

1901 MediPark Dr. #2050, Amarillo, Texas, 79106

PROTOCOL	CT ABDOMEN
Indication	ROUTINE
SERIES #	1
PARAMETERS	
Slice thickness(mm)	5
Index (interval)	5
FOV	PT SIZE
Algorithm	STD
CONTRAST	
Oral	YES
IV	YES
Amount	150CC
Bolus	3CC/S
Delay/trigger	60 SEC
SCANNING	
Start level	DOME OF DIAPHRAGM
Stop level	ILIAC CREST
Inspiration	YES
Expiration	NO
WINDOW RECONS	
Soft tissue	YES
Lung	YES
Bone	NO
Reformations	SAG AND CORONAL
NOTES	